


TOSI®-LumCheck Instructions for use

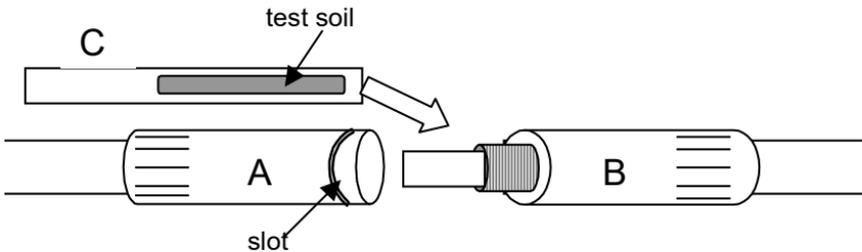

1. Unscrew LumCheck-device: detach part A (marked with slot) from part B (see picture).
2. Open protective pouch of TOSI®-LumCheck and insert the test object (part C) in part B as shown in picture. Do not touch the area covered with test soil.
3. Close LumCheck-device again.
4. Connect LumCheck-device with the rinse system of the washer (for example luer-lock) or reprocess it in the same way as done with cannulated instruments.
5. Open the device after the reprocessing cycle: disconnect part A from part B to remove the TOSI®-LumCheck without touching the test soil area!
6. For visual evaluation of the result, use the TOSI® evaluation table.

STORAGE:

Store in protective pouch at 2°C-25°C
Do not open protective pouch before use
Keep away from heat, light and chemicals

Shelf life: see imprint

CONTENT: 25 tests

Marketing & Sales:

Manufacturer:
PEREG GMBH
Porschestra. 12
D-84478 Waldkraiburg
Germany