


## Instructions for use

# SonoCheck

SonoCheck is a dosimeter used for monitoring the Ultrasonic energy.

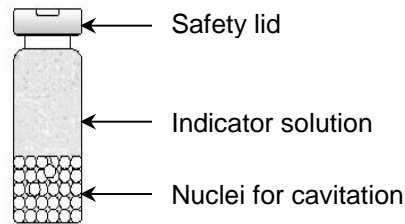

1. For the routine or validation test of the Ultrasonic cleaner, SonoCheck is placed in the ultrasonic basket. For detailed information regarding the functional test and the routine test please refer to: **The clean monitoring user guide**
2. Ultrasonic Cleaner is then switch on according to the CSSD manual procedure.
3. At the end of the cycle the vials are visually checked for the colour change from blue green to yellow indicating a positive result. NOTE: Strong ultrasonic energy can lead to a clear SonoCheck which is also a positive result.
4. The time needed for the colour change will provide additional information regarding the ultrasonic energy.
5. The results are then recorded in the **SonoCheck log book**.
6. In case of unsatisfactory results please refer to the **troubleshooting guide**.

Note: Do not use SonoCheck for washer-disinfectors because of the high temperature during drying cycle.

STORAGE: Store SonoCheck-box at 2°C - 25°C and do not freeze  
Keep away from light, heat and chemicals.

SHELF LIFE: see imprint

CONTENT: 30 tests

### Marketing & Sales:

**Manufacturer:**  
PEREG GMBH  
Porschestr. 12  
D-84478 Waldkraiburg  
Germany