


TOSI - Instructions for use

Monitoring of the cleaning efficiency of washer-disinfectors.

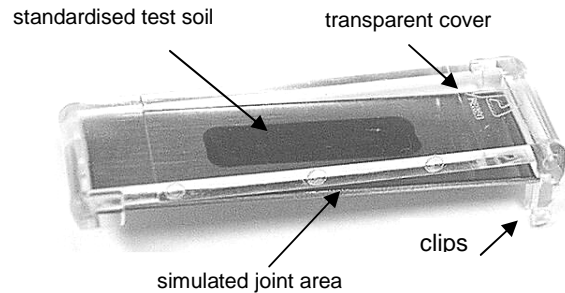

1. For the functional, validation or routine test of the cleaning efficiency, TOSI's are clipped flat on the individual instrument trays. For detailed information please refer to: **The clean monitoring user guide.**
2. To avoid any of the TOSI from being displaced by water pressure, make sure all TOSI's are safely clipped onto the wire trays or use the mini-trays if necessary.
3. The washer-disinfector is switched on according to the designated program.
4. At the end of the cycle the TOSI's are removed and visually checked for residues. A positive result (0) is indicated by a completely cleaned TOSI
5. In case of unsatisfactory results, please refer to the **troubleshooting guide.**
6. Results should be recorded in the TOSI **log book** or according to the department protocol.

STORAGE: Store in closed protective pouch in TOSI box at 2°C - 25°C.
Keep away from light, heat and chemicals.
Shelf life: see imprint.

CONTENT: 12 tests.

Marketing & Sales:

Manufacturer:

PEREG GMBH
Porschestr. 12
D-84478 Waldkraiburg
Germany